





- 1** Open the InPen app, click "Settings" and select "Sync Health Data"

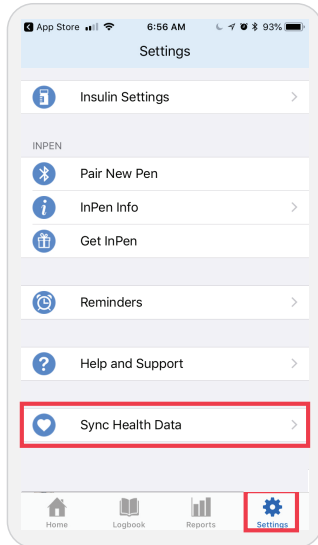


- 2** Click "Connect to Health"

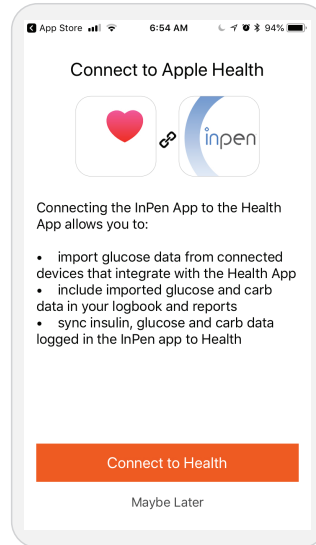


- 3** Click "Turn All Categories On" & "Allow"

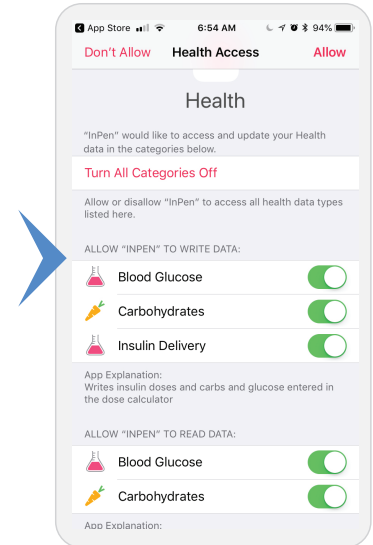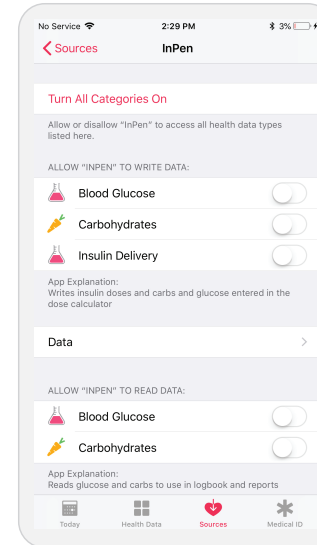

- 4** Open the glucose meter or Bluetooth® meter app. Under "Settings" look for "Health" or "Connections"

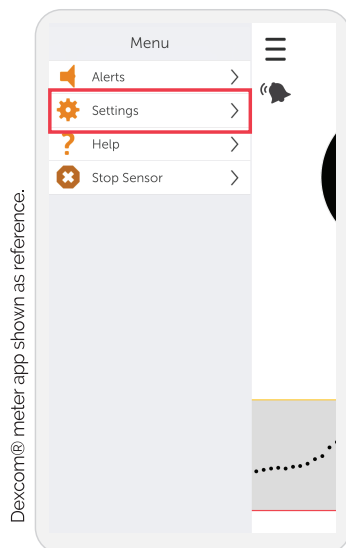

- 5** Connect to Apple Health, turn all categories on and click "Allow"

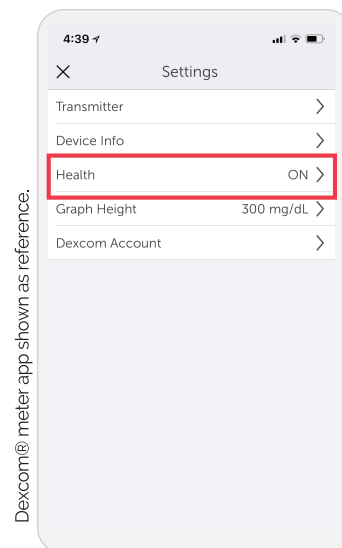

Bluetooth® Devices Compatible with Apple Health:

Accu-Chek® Aviva Connect	Accu-Chek® Guide
AgaMatrix Jazz Wireless 2	Contour Next One
Dexcom G5®	Dexcom G6®
iHealth Smart	One Drop Chrome
One Touch Verio® Flex	One Touch Verio® Sync

Requires iOS 10.0 or later. Compatible with iPhone, iPad, and iPod touch.

To learn more visit:
www.CompanionMedical.com



RX only. The Bluetooth® Wordmark and logos are owned by Bluetooth® SIG, Inc. and any use of such marks by Companion Medical, Inc. is under license. InPen logo is a trademark of Companion Medical. Copyright © 2019 Companion Medical and/or its affiliates. All rights reserved.