

How to push images from qpath to
epic

Order “qpath” in epic, turn on US machine

Routing Dx Association Edit Multiple

Order mode: Standard Providers

Currently Active Orders

Imaging

Active Discontinue Modify

US DOPPLER ARTERIAL LEG UNILAT

STAT w/ WET READ, 1 TIME IMAGING, Tue 10/22/13 at 0931, For 1 occurrence
R/O recurrence of femoral artery aneurysm in L groin.
Indicate right, left, or both: Left
Reason for exam: Aneurysm
Should this exam be performed portable? No

Accept Changes Discontinue All Active Orders

ED Orders

+ Search Pref List

QPATH ED BEDSIDE ULTRASOUND

STAT w/ WET READ, ONCE First occurrence Today at 0935

Prescriptions/Referrals

+ Search Pref List


Pharmacy Rx OTHER (NON E-PRESCRIBING PHARMACY) [Patient Preferred] None

Routing Dx Association Edit Multiple


Order mode: Standard Providers

Close F9

Wait for it...a percentage and the wifi icon will appear when worklist is ready

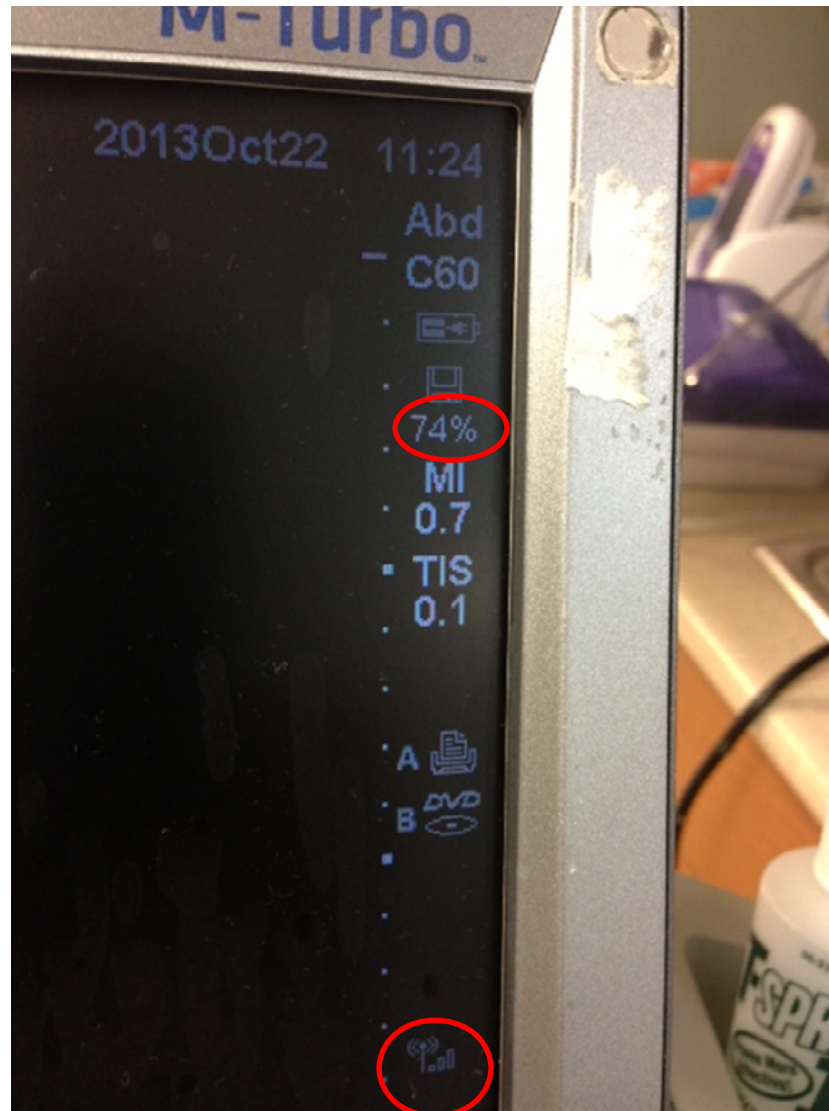

Hit the “patient” key

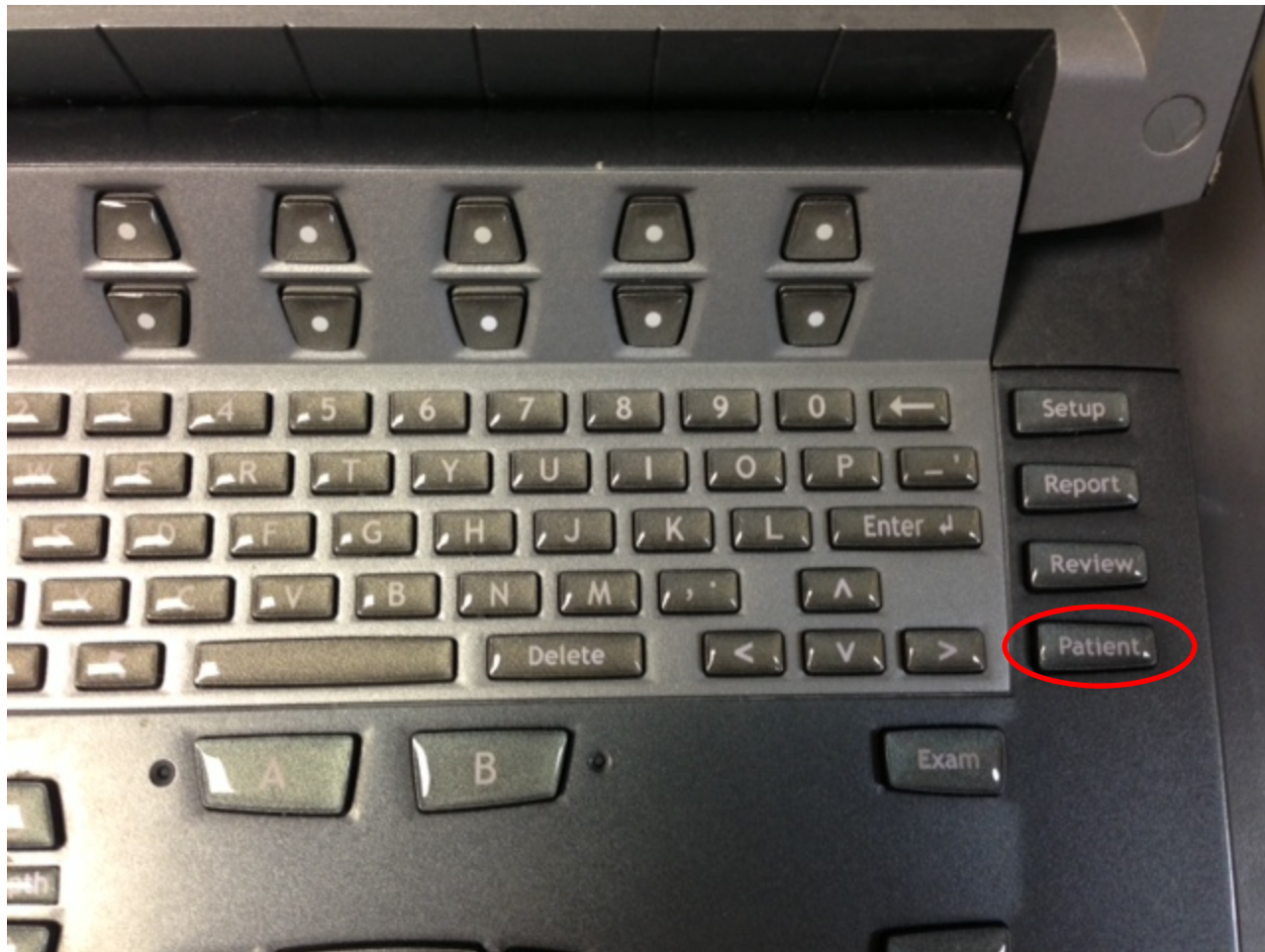


On the turbo/micromaxx, Hit
“worklist”

2013 Oct 22 09:37

Patient	Exam
Last	Type Vascular(Vas)
First	BP /
Middle	HR bpm
ID	
Accession	
Date of birth / / YYYY MM DD	
Gender	Reading Dr.
Indications	Referring Dr.
	Institution
User Procedure...	Symbols...
New/End Query Worklist Cancel Done	

Hit “update”. If you don’t see your patient, hit “query” then try again

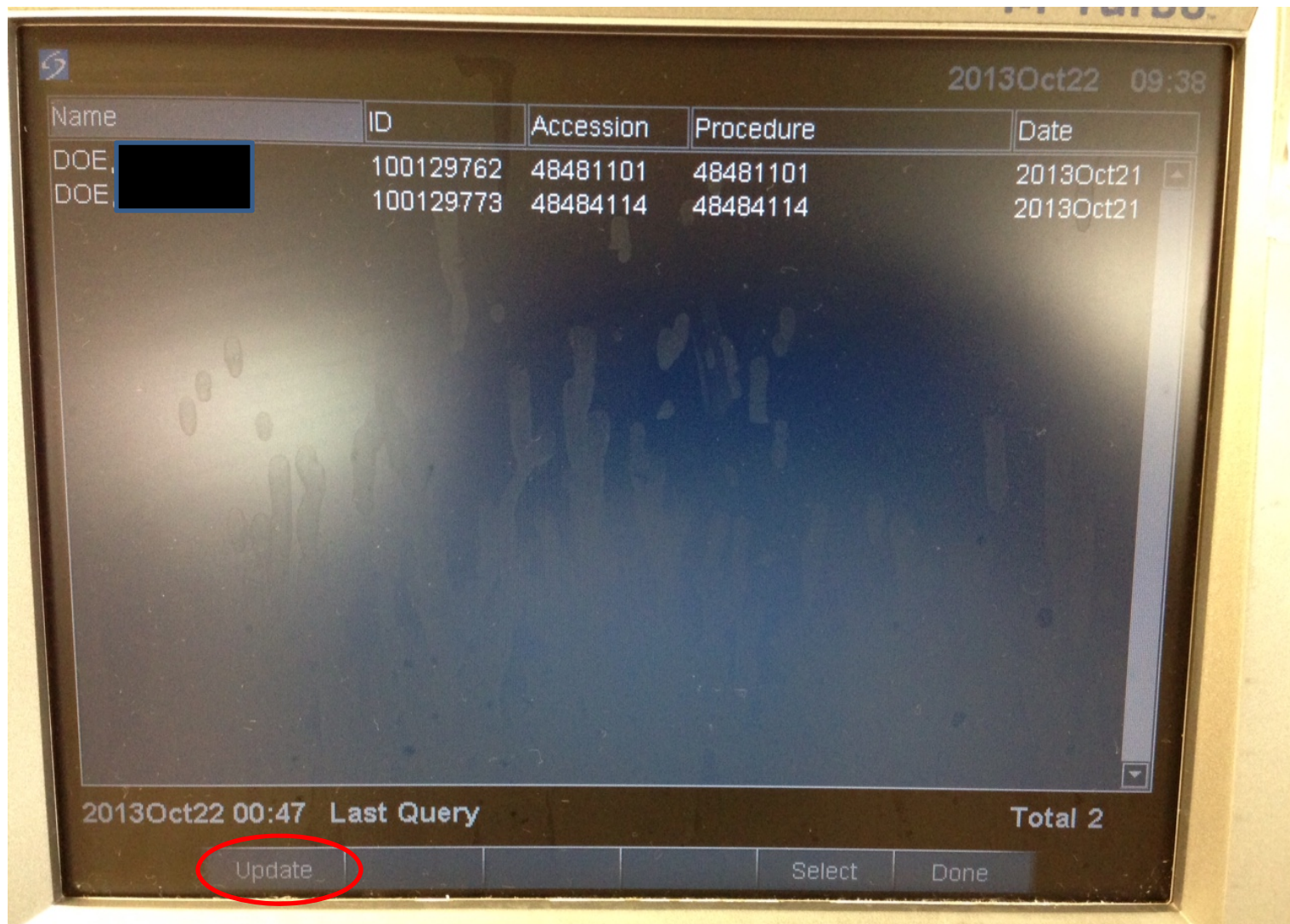

Your name in “reading dr”, attending
on “referring dr”.

The image shows the user interface of a SonoSite M-Turbo ultrasound machine. The screen displays patient and exam information. The 'Patient' section includes fields for Last (DOE), First (EQUAL), Middle, ID (100129762), Accession (48481101), Date of birth (1901 / 01 / 01), Gender (Female), and Indications. The 'Exam' section includes Type (Vascular(Vas)), BP, HR (bpm), Reading Dr. (circled in red), Referring Dr. (circled in red), and Institution. At the bottom, there are buttons for User, Procedure..., Symbols..., New/End, Worklist, Cancel, and Done.

Patient		Exam	
Last	DOE	Type	Vascular(Vas)
First	EQUAL	BP	/
Middle		HR	bpm
ID	100129762	Reading Dr.	
Accession	48481101	Referring Dr.	
Date of birth	1901 / 01 / 01	Institution	
Gender	Female		
Indications			


User: [] Procedure... Symbols... New/End Worklist Cancel Done

Don't forget to hit end after your exam


2013Oct22 09:37

Patient	Exam
Last	Type Vascular(Vas)
First	BP /
Middle	HR bpm
ID	
Accession	
Date of birth / / YYYY MM DD	
Gender	Reading Dr.
Indications	Referring Dr.
	Institution
User Procedure...	Symbols...
New/End Query Worklist X Cancel Done	


You can only push once per pt in epic-
send only the clinically relevant
images, split off the others

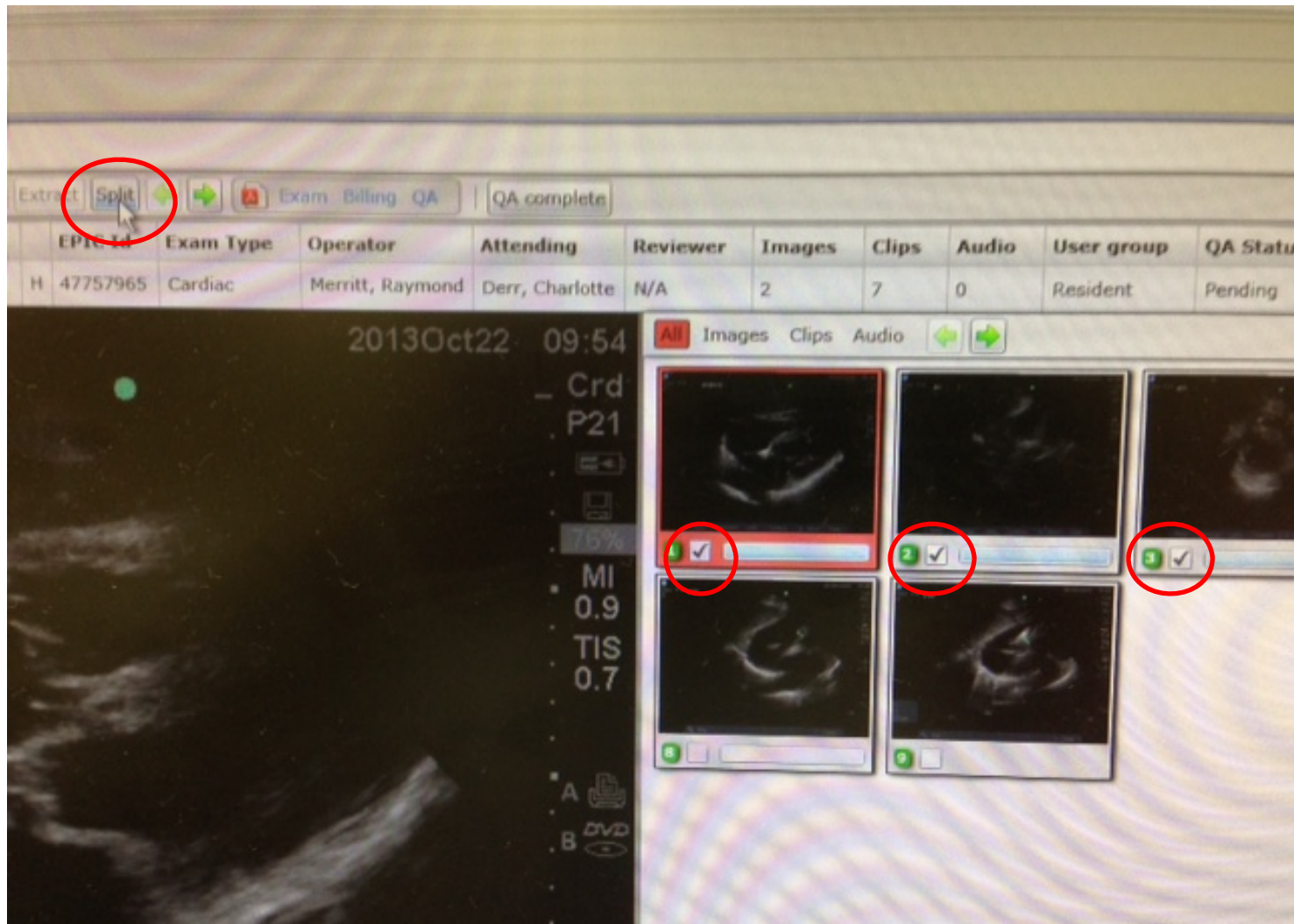


Select the patient then hit the
“transfer” key for only the exam you
want in EPIC

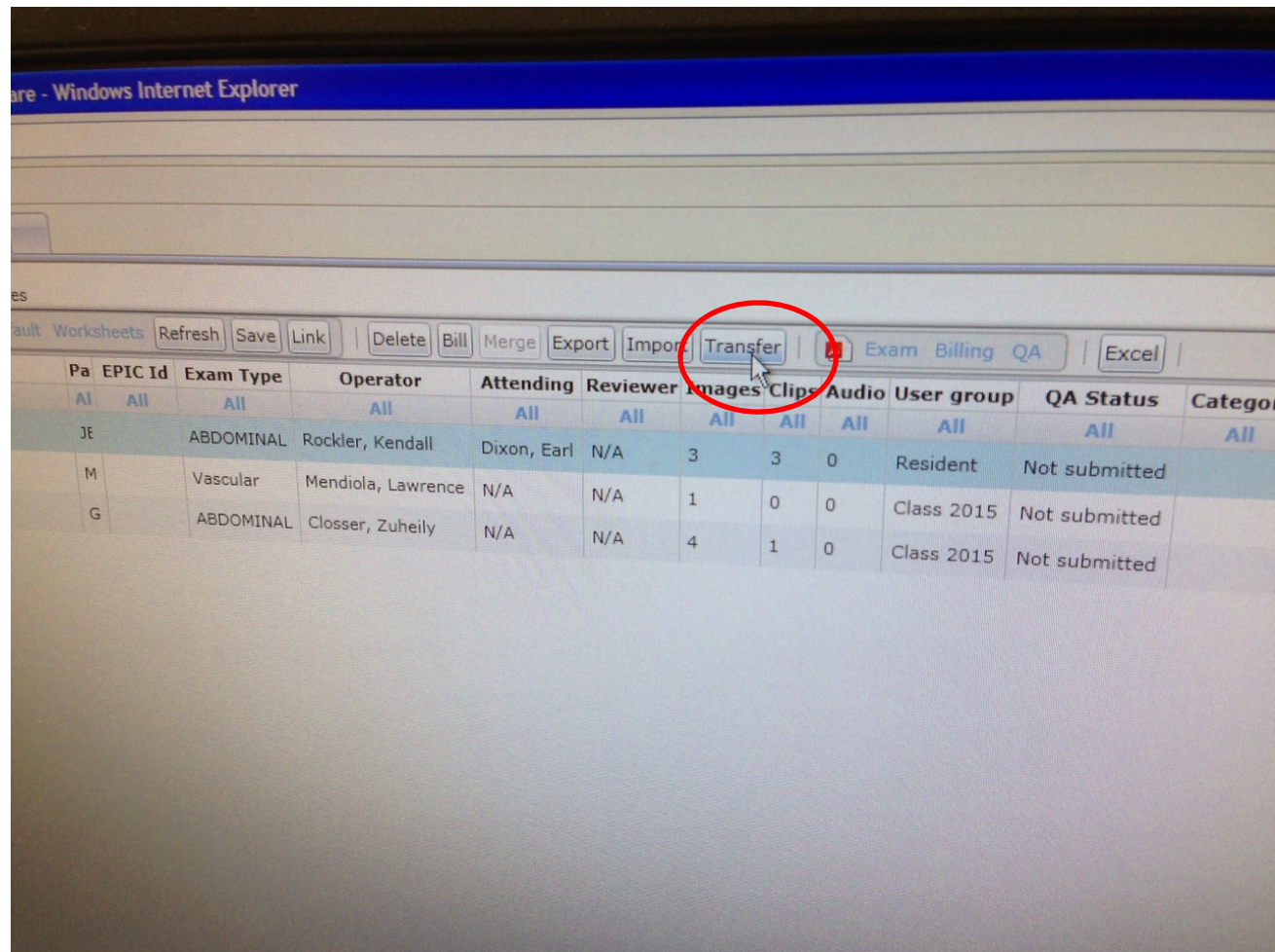


Hit “transfer”

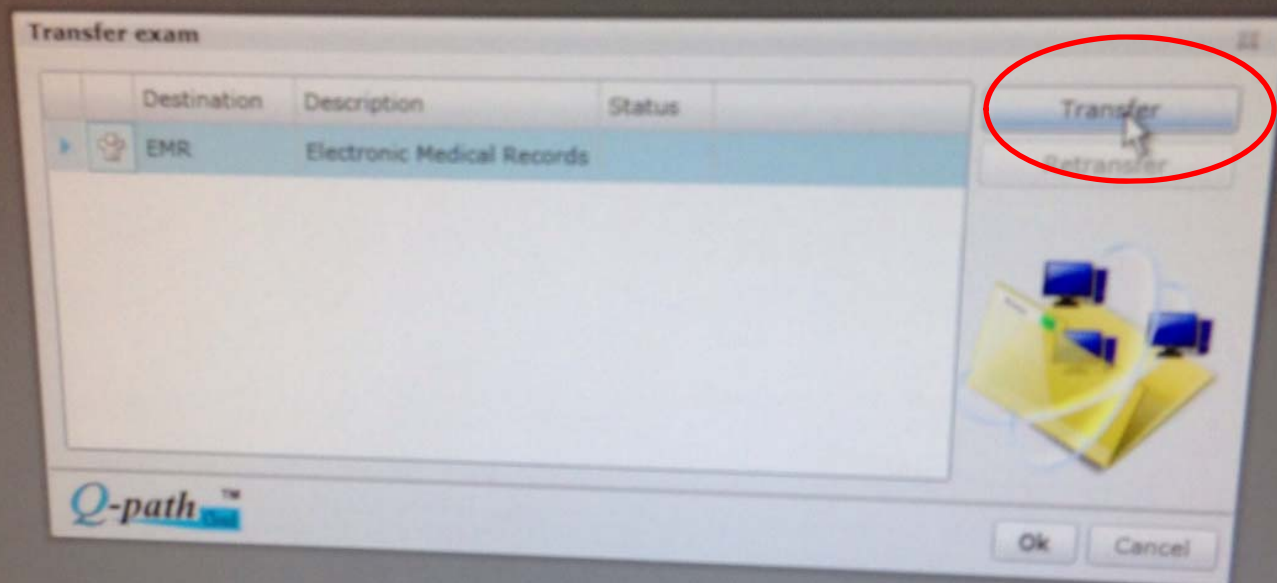

It will take you back to the main screen, notice the little man is yellow

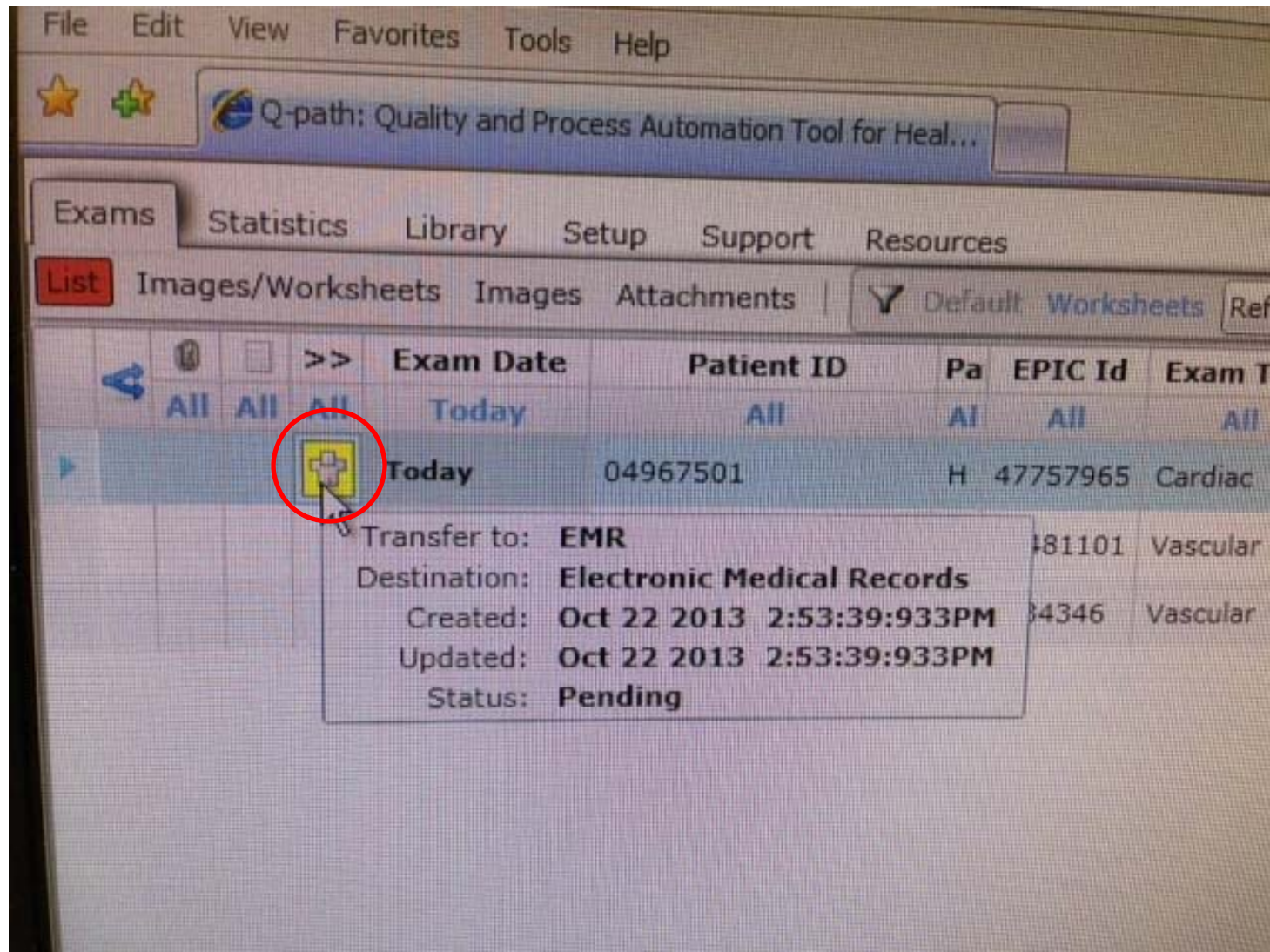

The little man will turn white when it has been completed

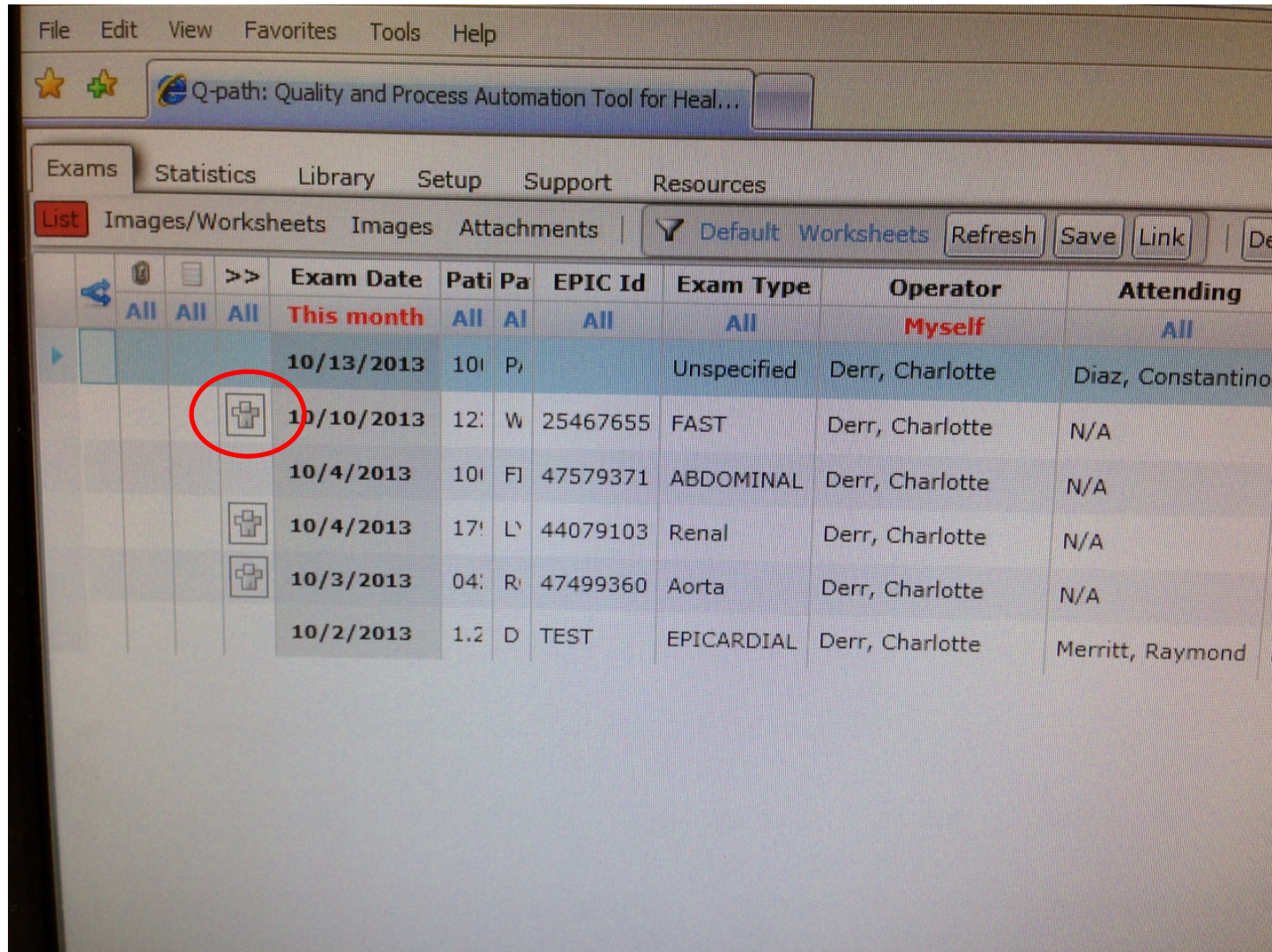

The screenshot shows a web application interface with a menu bar (File, Edit, View, Favorites, Tools, Help) and a browser address bar displaying 'Q-path: Quality and Process Automation Tool for Health...'. Below the menu bar, there are tabs for 'Exams', 'Statistics', 'Library', 'Setup', 'Support', and 'Resources'. Under the 'Exams' tab, there are sub-tabs for 'List', 'Images/Worksheets', 'Images', and 'Attachments'. A toolbar contains buttons for 'Default', 'Worksheets', 'Refresh', 'Save', and 'Link'. The main content area displays a table with the following columns: Exam Date, Patient, Patient ID, EPIC ID, Exam Type, Operator, and Attending. The table contains several rows of data, with the first row highlighted in blue. A red circle highlights a small icon in the first column of the table.

	Exam Date	Pati	Pa	EPIC Id	Exam Type	Operator	Attending
All	This month	All	AI	All	All	Myself	All
	10/13/2013	101	P		Unspecified	Derr, Charlotte	Diaz, Constantino
	10/10/2013	12	W	25467655	FAST	Derr, Charlotte	N/A
	10/4/2013	101	FI	47579371	ABDOMINAL	Derr, Charlotte	N/A
	10/4/2013	171	L	44079103	Renal	Derr, Charlotte	N/A
	10/3/2013	04	R	47499360	Aorta	Derr, Charlotte	N/A
	10/2/2013	1.2	D	TEST	EPICARDIAL	Derr, Charlotte	Merritt, Raymond

Hit discontinue to cancel the qpath order if you don't push images to epic.
The images will still be in qpath

